FAST PAD

SHAPE YOUR BODY

TONING - FIRING - SCULPTING - LIPOLYSIS - DRAINAGE

FAST PAD

The special bands FAST PAD, can be easily connected to the all GLOBUS electrostimulators with 2 or 4 channels and they have the same procedure of use as normal self adhesive electrodes.

INSTRUCTION FOR USE

- 1) Put a light coat of gel on the electrode surface (fig. 2).
- 2) Place the band on the areas to stimulate and block it by using the little elastic bands supplied with the product.
- 3) Connect each band and its cable to one of the electrostimulator channel using the supplied adapters (fig. 1).
- 4) Choose the program and start the stimulation.
- 5) At the end of the treatment, clean the contact surface with a wet duster (use only water) and dry it with care.

INDICATIONS

TONING
FIRMING
SCULPTING
LIPOLYSIS
DRAINAGE

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INDICATIONS

TONIFICATION
RAFFERMISSEMENT
SCULPTURE
LIPOLYSE
DRAINAGE

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FAST BAND

SHAPE YOUR BODY

TONING - FIRING - SCULPTING - LIPOLYSIS - DRAINAGE

FAST BAND

FAST BAND can be easily connected to all GLOBUS electrostimulators WITH 2 AND 4 channels.

This band has been studied to perform with great easiness and efficacy all the stimulation programs for abdominal areas (abdominal muscles and oblique) and gluteus.

Placing the electrodes directly on the lumbar zone, FAST BAND also allows to apply the TENS currents that help you to eliminate backpain

Before wearing the band for treatment, it is necessary to dampen the rubber electrodes with water or gel.
To carry out the treatment connect the cables following the figures below.

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